

TML\MSH Microbiology Department Policy & Procedure Manual	Policy # MI\VIR\17\09\v02	Page 1 of 1
Section: Serology Manual	Subject Title: Appendix IX Shipment of Parvovirus PCR to HSC	
Issued by: LABORATORY MANAGER	Original Date: Dec 20,2003	
Approved by: Laboratory Director	Revision Date:	

APPENDIX IX

SHIPMENT OF PARVOVIRUS PCR TO HSC

1. Fill up a HSC requisition.
2. Put specimen and requisition in biohazard bag, and then in a brown paper bag.
3. Put 'Hospital for Sick Children, Microbiology Receiving, 3rd Floor Atrium, Rm3676' sticker on brown bag.
4. Put labeled brown bag into blue specimen container, and send to 'TG specimen management'.
5. Enter in LIS as 'specimen send to HSC' and finalize.
6. Also enter in serology send out binder.