TML\MSH Microbiology Department	Policy # MI/SER/17/03/v01	Page 1 of 1
Policy & Procedure Manual		
Section: Serology Manual	Subject Title: Appendix III	
	Shipment of JC/BK Parvovirus PCR	
	Samples	
Issued by: LABORATORY MANAGER	Original Date: March 14, 2001	
Approved by: Laboratory Director	Revision Date: September 20, 2001	

APPENDIX III

SHIPMENT OF JC/BK PCR SAMPLES

PHL requires that these samples be packaged in a standardized way before they will forward them to Hamilton for us:

- 1. Put sample tube inside plastic bag and place together with requisition into a plastic cylinder.
- 2. Place plastic cylinder into a cardboard box, padded with paper towels.
- 3. Seal the box with packing tape and label the box as follows:

Public Health Laboratory, 81 Resources Rd.

DO NOT OPEN

Please forward to:

JC/BK PCR
Dr. James Mahoney,
REGIONAL VIROLOGY LABORATORY,
ST. JOSEPH'S HOSPITAL,
50 CHARLTON AVE., E.,
HAMILTON, ONTARIO
L8N2X6