

TML/MSH Microbiology Department Policy & Procedure Manual	Policy #MI\VIT\11\01\v01	Page 1 of 1
Section: Vitek Manual	Subject Title: Mixed Cultures But Results Has Been Transferred	
Issued by: LABORATORY MANAGER	Original Date: October 4, 1995	
Approved by: Laboratory Director	Revision Date: May 30, 2001	

MIXED CULTURES BUT RESULTS HAVE BEEN TRANSFERRED

DELETE card results from the transferred data when purity plate shows a mixed culture:

1. At Main Menu, click "**DAILY**"
2. click "**ENTER PATIENT INFO**"
3. click "**LAB #**" box
4. enter Lab #
5. click "?" for Lookup Table
6. select appropriate record when Lookup Table appears, click "**OK**"
7. click "**CARD**" (at the bottom of the screen)
8. click the card to be deleted from the list
9. click "**SELECT**" box and enter isolate #
10. click "**DELETE**"
11. click "**FILE**"
12. click "**QUIT**"
13. click "**FILE**"
14. click "**QUIT**"