

TML/MSH Microbiology Department Policy & Procedure Manual	<b>Policy #MI\VIT\17\01\v01</b>	Page 1 of 1
Section: <b>Vitek Manual</b>	Subject Title: <b>Reboot Instrument</b>	
Issued by: <b>LABORATORY MANAGER</b>	Original Date: October 4, 1995	
Approved by: Laboratory Director	Revision Date: May 30, 2001	

### **REBOOT INSTRUMENT**

From Main Menu:

1. Click "**System**"
2. Click "**System Maintenance**"
3. Click "**Stop the System**"
4. Click "**Execute**"
5. Press the on/off button on the Vitek CPU to turn off.
6. Wait 30 seconds.
7. Press the on/off button on the Vitek CPU to turn on.
8. When Login/Password prompts appear, type in Login: SUPV  
Password: SUPV